
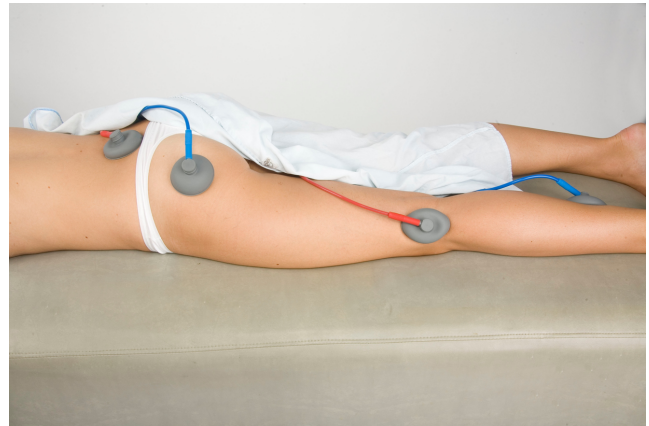


# Treatment Plan Example Flowchart NeuroMed - *MATRIX System*<sup>TM</sup>

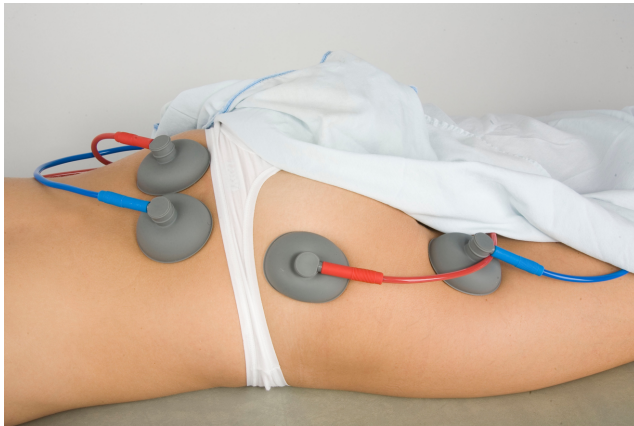

- **Severe and/or Neurologic Sciatica Pain**

1<sup>st</sup> week – (5 treatments) treat every day – **A**

- Program **01** / 4 electrodes with PS (scan), Set dosage slowly to patients tolerance level – Then use program **01** / 2 electrodes, no PS (scan), Set dosage slowly to patients tolerance level - After 5 treatments re evaluate patient

---

- **Sciatica Pain with Muscle Spasm**

(5 treatments) treat every other day – **B**

- Program **31** / 4 electrodes with PS (scan), Set dosage slowly to above sensation level – Then use program **01** / 2 electrodes, no PS (scan), Set dosage slowly to patients tolerance level - After 5 treatments re evaluate

---

(Next 5 treatments) treat every other day - **C**



- If the patient had **acute pain condition** use Program **03** / 4 electrodes with PS (scan), Set dosage slowly to above sensation level – Then use program **01** / 2 electrodes, no PS (scan), Set dosage slowly to patients tolerance level - After 5 treatments re evaluate
- If the patient had **chronic pain condition** use Program **05** / 4 electrodes with PS (scan), Set dosage slowly to above sensation level – Then use program **01** / 2 electrodes, no PS (scan), Set dosage slowly to patients tolerance level - After 5 treatments re evaluate

---

Maintenance programs / Treat 1 or 2 times per month - **D**



- Program **09** / 4 electrodes with PS (scan), Set dosage slowly to above sensation level – Then use program **01** / 2 electrodes, no PS (scan), Set dosage slowly to patients tolerance level