



# Treatment Plan Example Flowchart NeuroMed - *MATRIX System*™

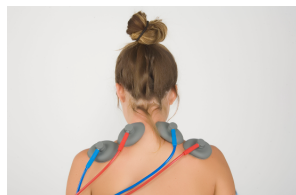

or

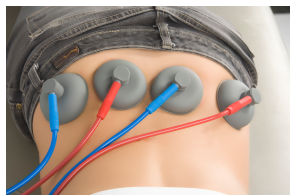

or



## • *Electroanalgesia and Decompression Treatments*

### Then next 2 patient treatments Start with

- Program **31 / 4 electrodes** with PS (scan), Set dosage slowly to patients sensation level – Then use program **01 / 4 electrodes**, no PS (scan), Set dosage slowly to patients tolerance level



- Program **27 / 4 electrodes** with PS (scan), Set dosage slowly to patients sensation level – Then use program **01 / 4 electrodes**, no PS (scan), Set dosage slowly to patients tolerance level - For 2 patient treatments



- Program **06 / 4 electrodes** with PS (scan), Set dosage slowly to patients sensation level – Then use program **01 / 4 electrodes**, no PS (scan), Set dosage slowly to patients tolerance level - For 2 patient treatments



- Program **32 / 4 electrodes** with PS (scan), Set dosage slowly to patients sensation level – Then use program **01 / 4 electrodes**, no PS (scan), Set dosage slowly to patients tolerance level - For 2 patient treatments



- Program **34 / 4 electrodes** with PS (scan), Set dosage slowly to patients sensation level – Then use program **01 / 4 electrodes**, no PS (scan), Set dosage slowly to patients tolerance level - For 2 patient treatments



Repeat Treatment Plan from the top



Maintenance programs / Treat 1 or 2 times per month -

- Program **09 / 4 electrodes** with PS (scan), Set dosage slowly to above sensation level – Then use program **01 / 4 electrodes**, no PS (scan), Set dosage slowly to patients tolerance level