


Treatment Plan Example Flowchart NeuroMed - *MATRIX*

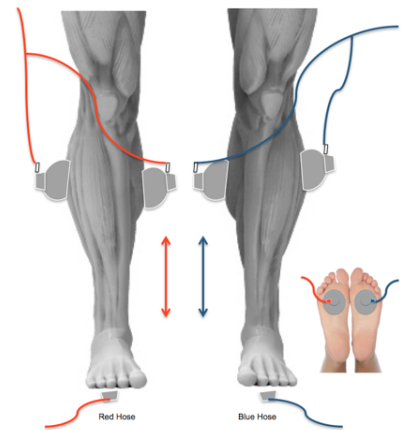

• Neuropathy with Severe and/or Neurologic Pain

Treat every other day – **A**

- Program **10 / 6 electrodes** with PS (scan), Set dosage slowly to above sensation level (20 – 30 Ma) – Then use program **01 / 6 electrodes**, with PS (scan), Set dosage slowly to patients tolerance level (35-40 Ma) - After 5 -10 treatments re evaluate patient

• Neuropathy with no Pain

Treat every other day – **A1**

- Program **10 / 6 electrodes** with PS (scan), Set dosage slowly to above sensation level (20 – 30 Ma) – Then use program **10 / 6 electrodes**, with PS (scan), Set dosage slowly to above sensation level (35-40 Ma) - After 5 -10 treatments re evaluate patient

If no positive response, after 10 treatments, try this treatment setup:



• Pain



Treat every other day – **B**

- Program **31 / 4 Large pad electrodes** with PS (scan), Set dosage slowly to above sensation level – Then use program **01 / 4 large pad electrodes**, with PS (scan), Set dosage slowly to patients tolerance level - After 5 -10 treatments re evaluate patient

• No Pain

Treat every other day – **B1**

- Program **10 / 4 Large pad electrodes** with PS (scan), Set dosage slowly to above sensation level – Then use program **31 / 4 large pad electrodes**, with PS (scan), Set dosage slowly to above sensation level - After 5 -10 treatments re evaluate patient

Maintenance programs / Treat 1 or 2 times per month - **C**



- Program **09 /** Apply electrodes with PS (scan), Set dosage slowly to above sensation level – Then use program **01 /** apply electrodes, with PS (scan), Set dosage slowly to above sensation level.