


Treatment Plan Example Flowchart NeuroMed – MATRIX System™

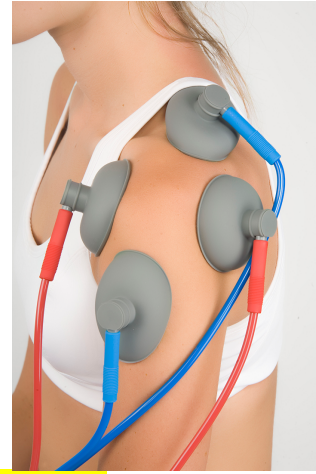

- **Severe and/or Neurologic Pain Joint Pain** -

1st week – (5 treatments) treat every day – **A**

- Program **01 / 4 electrodes** with PS (scan), Set dosage slowly to patients tolerance level – Then use program **01 / 2 electrodes**, no PS (scan), Set dosage slowly to patients tolerance level - After 5 treatments re evaluate patient

- **Joint Pain**

(5 treatments) treat every other day – **B**

- Program **14 / 4 electrodes** with PS (scan), Set dosage slowly to above sensation level – Then use program **01 / 2 electrodes**, no PS (scan), Set dosage slowly to patients tolerance level - After 5 treatments re evaluate

- **Post Trauma Joint Pain**

(5 treatments) treat every other day – **A1**

- Program **27 / 4 electrodes** with PS (scan), Set dosage slowly to above sensation level – Then use program **01 / 2 electrodes**, no PS (scan), Set dosage slowly to patients tolerance level - After 5 treatments re evaluate

(Next 5 treatments) treat every other day - **C**



- If the patient had **acute pain condition** use Program **03 / 4 electrodes** with PS (scan), Set dosage slowly to above sensation level – Then use program **01 / 2 electrodes**, no PS (scan), Set dosage slowly to patients tolerance level - After 5 treatments re evaluate
- If the patient had **chronic pain condition** use Program **05 / 4 electrodes** with PS (scan), Set dosage slowly to above sensation level – Then use program **01 / 2 electrodes**, no PS (scan), Set dosage slowly to patients tolerance level - After 5 treatments re evaluate

Maintenance programs / Treat 1 or 2 times per month - **D**



- Program **09 / 4 electrodes** with PS (scan), Set dosage slowly to above sensation level – Then use program **01 / 2 electrodes**, no PS (scan), Set dosage slowly to patients tolerance level