


# Treatment Plan Example Flowchart - Headaches



## NeuroMed - *MATRIX System*™

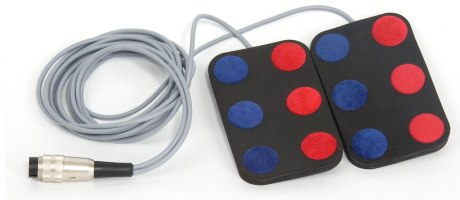

### • Migraine

(5 treatments) treat every other day –

- Program **07** / 6 Field electrodes with PS (scan), Set dosage slowly to above sensation level – Then use program **01** / 6 Field electrodes, no PS (scan), Set dosage slowly to above sensation level - After 5 treatments re-evaluate patient

### • Cluster

•


(5 treatments) treat every other day – Pad placement (same as above)

- Program **23** / 6 Field electrodes with PS (scan), Set dosage slowly to above sensation level – Then use program **01** / 6 Field electrodes, no PS (scan), Set dosage slowly to above sensation level - After 5 treatments re-evaluate patient

### • Tension

(5 treatments) treat every other day –

- Program **27** / 4 electrodes with PS (scan), Set dosage slowly to above sensation level – Then use program **01** / 4 electrodes, no PS (scan), Set dosage slowly to above sensation level – After 5 treatments re-evaluate

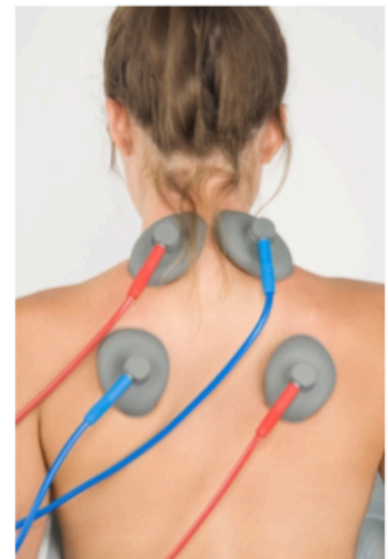

Maintenance programs / Treat 1 or 2 times per month -



- Program **09** / 4 electrodes with PS (scan), Set dosage slowly to above sensation level – Then use program **01** / 2 electrodes, no PS (scan), Set dosage slowly to patients tolerance level